



**PLEASE NOTE THE MEDICATION CUP IS A REPLACEABLE PART.
ITS LIFE IS VERY DEPENDENT ON THE DRUG NEBULISED AND HOW WELL IT IS MAINTAINED.**



This chart is meant to be used as a guide for you and your veterinarian. Some of the medications listed are not available in all countries. It is recommended to always consult your veterinarian for advice regarding diagnosis and the treatment protocol for your horse.

Antibiotics

Drug	Trade Name	Drug Type	Drug Format	Delivery time for 1ml	Dilute with Saline
Cefquinome	Cobactan®	Cephalosporins	Cobactan® 4.5% Injectable Solution	1.5 to 3 Minutes	No
Ceftiofur Sodium	Excenel®	Cephalosporins	Excenel® 4G Injectable Solution	1.5 to 3 Minutes	No
Gentamicin	Gentaject	Aminoglycosides	Gentaject 10% Injectable Solution	1.5 to 3 Minutes	Yes 1:1

Bronchodilators

Drug	Trade Name	Drug Type	Drug Format	Delivery time for 1ml	Dilute with Saline
Ipratropium	Atrovent®	Anticholinergic Agent	Atrovent® 250 UDVS Nebuliser solution	0.5 to 1 minute	Yes 1:1
Salbutamol (Albuterol)	Ventolin™	B2-adrenergic receptor agonist	Ventolin™ 2.5 Nebuliser Solution	0.5 to 1 minute	Yes 1:1
Clenbuterol	Ventipulmin®	B2-adrenergic receptor agonist	Ventipulmin® 30 Injectable Solution	0.5 to 1 minute	Yes 1:1

Corticosteroids

Drug	Trade Name	Drug Type	Drug Format	Delivery time for 1ml	Dilute with Saline
Dexamethasone	Dexameth	Corticosteroid	Dexameth 2mg/ml Injectable Solution	0.75 to 1.5 Minutes	Yes 1:1
Fluticasone Propionate	Flixotide™	Corticosteroid	Flixotide™ 2mg/2ml Nebuliser Suspension	0.75 to 1.5 Minutes	Yes 1:1
Budesonide	Pulmicort®	Corticosteroid	Pulmicort® 0.5 Nebuliser Solution	1 to 2 Minutes	Yes 1:1

Mucolytics

Drug	Trade Name	Drug Type	Drug Format	Delivery time for 1ml	Dilute with Saline
Acetylcysteine	Parvolex®	Mucolytic	Parvolex® 2g Solution for infusion	1.75 to 3.5 Minutes	Yes 1:1
0.9% Saline Solution	0.9% Saline Solution	Mucolytic	B.Braun NaCl 0.9%	1 Minute or Less	No

Other Therapies

Active Ingredients	Trade Name	Notes
Grindelia, Water and Essential Oils	Balsamic Air	A natural therapy. Follow supplier instructions for use. It is recommended to dilute with saline 1:1 for use with Flexineb.
Chelated Silver and Distilled Water	SilvaPlex™ and EquiSilver™	Veterinary Antimicrobial Silver Respiratory Solution. Follow supplier instructions for use.
Natural hydro-soluble extract of Curcuma Longa	Nebulin®	A natural therapy. Follow supplier instructions for use.